




Instructions for Use for B&D

ST-MS-WI-07
REV.00

VALID

History of changes

Revision No.	Date of issue	Reason for change	Summary of change	Provided by	Approved by
00	27/04/2020	NA	NA	Aryan Deldari 	Saeed Nazari 

VALID

--	--	--	--	--	--

Instructions for use:

- 1) Daily, place the Test Pack on the bottom shelf, over the drain closest to the door, in an otherwise empty chamber.
- 2) Process in a pre-vacuum cycle of 134°C for 3.5 or 121°C for 15 minutes exposure time.
- 3) After processing, carefully remove the Test Pack and allow to cool.
- 4) Confirm that the Indicator dot on the exterior label of the Test Pack has changed color from Blue to Pink and then remove the internal Indicator sheet from the center of the Test Pack.
- 5) The Indicator sheet should be a uniform dark color. Inconsistent or color change which is not uniform may indicate sterilizer malfunction and should be immediately reported.
- 6) Complete the information on the Indicator sheet and retain as permanent record.

The chemical reaction which causes the color transition is a steam specific reaction and is irreversible

Safety Recommendations

- If there is any doubt about sterilization, the tool should be considered unsterile.
- Do not use the indicator which are not brown before using in the sterilization cycle.
- It is not appropriate to use flash in the sterilization cycle.

Storage

- Should be kept at room temperature (0 to 40 ° C) and humidity of 10 to 70%, away from oxidizing agents and fluids.
- Keep unused indicators in the provided bag.
- Store away from direct sunlight.