



Instructions for Use for type 4

ST-MS-WI-08
REV.00

Instructions for use:

1. Prepare Package for sterilization regarding the standard procedure that being used in your CSSD.
2. Put the indicator tests in each pack and prepare them for sterilization. Place the indicator in the farthest point to the steam inlet in smaller packs and in different parts of the pack in larger ones.
3. Sterilize the pack according to the standard sterilization method.
4. After processing, the dark color should have entered the ACCEPT window of the type 5 Steam Chemical Integrator. If the dark color has not entered the ACCEPT window, this indicates a REJECT result which means that the items in the pack, peel pouch, container system, or tray were not exposed to sufficient steam sterilization conditions. These items should be returned for reprocessing.
5. Stated Value: -25% Parameter: Temperature, time, saturated steam
6. Attach indicator to the patient documents file or to your sterilization process history documents.

Safety Recommendations

- If there is any doubt about sterilization, the tool should be considered unsterile.
- Do not use the indicator which are not brown before using in the sterilization cycle.
- It is not appropriate to use flash in the sterilization cycle.

Storage

- Should be kept at room temperature (0 to 40 ° C) and humidity of 10 to 70%, away from oxidizing agents and fluids.
- Keep unused indicators in the provided bag.
- Store away from direct sunlight.